



# Foldable Toilet Foot Perch



Thank you for purchasing our Foldable Toilet Foot Perch, a raised footrest that will help you relieve painful straining on the toilet.

We hope you will be happy with your purchase.

## WARNING:

- Read the instruction sheet carefully before using the Foldable Toilet Foot Perch
- DO NOT STAND ON THE FOLDABLE FOOT PERCH AT ANY TIME
- THE FOLDABLE FOOT PERCH SHOULD NOT BE USED AS A STEP STOOL
- Only use on a flat, stable non-slip surface
- Not to be used in a shower / bath or any other wet surfaces
- MAXIMUM WEIGHT LIMIT: 80KG

## Instructions:


1. Unfold front support from the closed position so it is out of the way.



2. Unfold the main support legs.



3. Fold the front support back towards the main support legs and push to snap into place.

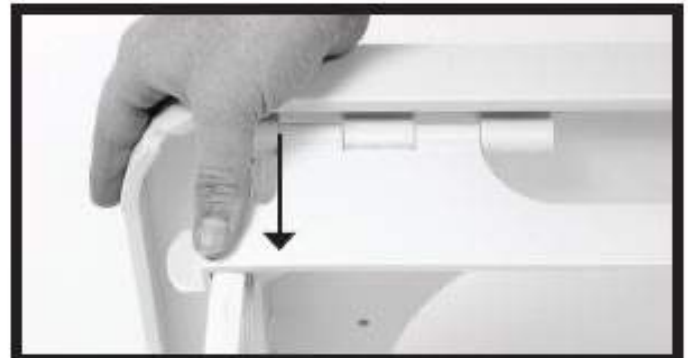

BEFORE USING THE PRODUCT, MAKE SURE THE FRONT SUPPORT IS CORRECTLY SNAPPED INTO PLACE TO AVOID DANGER OF COLLAPSE

Storage: Simply slide the Foldable Toilet Foot Perch around the toilet when not in use

