Leg and Foot stimulator

Item codes: M533 / 590580

Instructions for Use

v001: 19/12/23

Please read and retain these instructions for future reference

ENGLISH INSTRUCTIONS

Power details and Technical data:

Model No: KMS008M

Max. power: 30W

Input: 100-240V, 50/60Hz

Adapter output: 24V 1500mA

This product must be kept dry and not exposed to direct sunlight.

Please read this instruction manual thoroughly before starting and ensure you are familiar with the operation of your new Leg and Foot Stimulator.

OPERATION PANEL AND REMOTE CONTROL BUTTONS:

Power button: Press this button on device to turn the unit ON or OFF.

Start/Stop: Press this button to start with your chosen settings. When the device is in operation, press this button to stop it.

Time-Reduce the time. The pre-set time is 15 minutes, press this button to reduce the time.

Time+: Increase the time. Press this button to increase the time. The maximum adjustable time is 30 minutes.

Speed+: Increase the speed. The speed can be manually set from 1 to 10. Press this button to increase the speed while the device is running.

Speed-: Reduce the speed. The speed can be manually set from 1 to 10. Press this button to reduce the speed while the device is running.

Program Button: You could change programs between manual mode and automatic mode via this button. By pressing repeatedly, you can switch between the programs, HO, UI, P1, P2 and P3. When you press Program button on time, “UI” appears on the display. In this mode, with oscillation function only and customizable speed. In the P1 program, the oscillation and massage function are active, and the device runs through a program with pre-set speed range. You can not change the speed in this program. In the P2 program, the device will switch between oscillation and massage with the pre-set time and speed range. In the program P3, the acupressure heads are deactivated and only the oscillation is activated, with the pre-set speed range.

Manual mode “HO”: In standby mode, when you press “Start/stop” button, the device will in manual mode, “HO” appears on the display, in this mode, the oscillation and massage functions run continuously, the speed is customizable.

In standby mode, you can adjust the time but not the speed. Once the device has been put into operation, you can adjust the speed but not the time.

Important notes on operation:

- You can select the P mode (Auto Program) after switching on the device; press Start/stop to activate the appliance.
- Stop the appliance by first pressing Start/stop. Then switch off the device with the power button or press Start/stop again to continue the massage.
- In manual mode “HO”, you can set the time before the massage starts and adjust the speed during the massage.

Note: In program P3, the rotating acupressure heads will stop as soon as you place your feet on the device. This is not a defect.

Table : Program overview of P1, P2, P3 and manual mode:

AUTOMATIC PROGRAM P1	
FUNCTION	SPEED RANGE (CHANGES PER MINUTE)
Oscillation and Acupressure	1-2-3-4-5-6-7-8-5-3-1-2-3-4-5

AUTOMATIC PROGRAM P2		
STANDARD TIME 15 MINUTES TOTAL	FUNCTION	SPEED LEVEL
1st minute	Oscillation & acupressure	1 level
2nd minute	Oscillation only	3 level
3rd minute	Oscillation & acupressure	5 level
4th minute	Oscillation only	7 level
5th minute	Oscillation & acupressure	9 level
6th minute	Oscillation only	10 level
7th minute	Oscillation & acupressure	8 level
8th minute	Oscillation only	6 level
9th minute	Oscillation & acupressure	4 level

10th minute	Oscillation only	2 level
11th minute	Oscillation & acupressure	1 level
12th minute	Oscillation only	3 level
13th minute	Oscillation & acupressure	5 level
14th minute	Oscillation only	7 level
15th minute	Oscillation & acupressure	9 level

AUTOMATIC PROGRAM P3	
FUNCTION	SPEED RANGE (CHANGES PER MINUTE)
Oscillation only	3-5-7-9-2-4-6-8-10-1-3-5-7-9-2

Manual Mode "HO" - Oscillation & acupressure	
Oscillation & Acupressure	Speed is adjustable

Manual Mode "UI"-Oscillation only	
Oscillation only	Speed is adjustable

User instructions:

The device is designed for use on the arches of the feet, on the legs, and on the back. Do not use upon the waist, neck or other parts of the body not described in this manual. Do not use the device for any purposes other than those specified in this manual.

1. Plug the power adapter into the power outlet and connect the cable to the unit. As soon as the device is connected to power, it will beep once and the display will light up.
2. Once the device is connected, turn it so that the display faces forward. Select a mode according to your requirements using the remote control or the buttons on the device. Your settings will be shown on the display. To start, press the Start/Stop button on remote control or on the device.
3. To switch off Leg and Foot Stimulator correctly, you must first stop the device by pressing the Start/stop button on the device or on the remote control. Then press the power button on the device to switch off the device completely.
4. Disconnect the appliance from the power outlet after use.

ATTENTION: For sensitive floor surfaces (eg.parquet) we recommend placing a film mat or carpet under the unit. This prevents damage to and discolouration of your flooring.

Switching off the Leg and Foot stimulator

- Switch off Leg and Foot stimulator correctly, you must first stop the device by pressing the stop button on the device or on remote control. Then press the power button to switch off the device completely.

WARNINGS AND SAFETY INSTRUCTIONS:

WHEN USING ELECTRICAL APPLIANCES, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED:

- READ ALL INSTRUCTIONS BEFORE USE.
- FOR DOMESTIC INDOOR USE ONLY.
- DO NOT USE OUTDOORS OR FOR COMMERCIAL USE.
- ONLY USE WITH THE MAINS ADAPTOR/CHARGER SUPPLIED.
- **DO NOT** USE ANY OTHER MAINS ADAPTOR/CHARGER AS THIS MAY CAUSE DAMAGE AND INVALIDATES ANY WARRANTY YOU MAY HAVE.
- BEFORE USE, CHECK THOROUGHLY FOR ANY DEFECTS AND DO NOT USE IF DEFECTS ARE FOUND. TAKE CARE NOT TO DROP THE APPLIANCE AS HEAVY IMPACTS MAY CAUSE INTERNAL DAMAGE.

- BEFORE CONNECTING THE APPLIANCE, CHECK IF THE VOLTAGE INDICATED ON THE BOTTOM OF THE APPLIANCE IS IN ACCORDANCE WITH THE MAINS VOLTAGE IN YOUR HOME.
- CHECK THE APPLIANCE AND POWER CABLE CAREFULLY FOR DAMAGE BEFORE EACH USE.
- THIS APPLIANCE CAN BE USED BY CHILDREN AGED FROM 8 YEARS AND ABOVE, AND PERSONS WITH REDUCED PHYSICAL, SENSORY OR MENTAL CAPABILITIES OR LACK OF EXPERIENCE AND KNOWLEDGE IF THEY HAVE BEEN GIVEN SUPERVISION OR INSTRUCTION CONCERNING USE OF THE APPLIANCE IN A SAFE WAY AND UNDERSTAND THE HAZARDS INVOLVED. CHILDREN SHALL NOT PLAY WITH THE APPLIANCE. CLEANING AND USER MAINTENANCE SHALL NOT BE MADE BY CHILDREN WITHOUT SUPERVISION.
- CHILDREN OF LESS THAN 3 YEARS SHOULD BE KEPT AWAY UNLESS CONTINUOUSLY SUPERVISED.
- **SOME PARTS OF THIS PRODUCT CAN BECOME VERY HOT AND CAUSE BURNS. PARTICULAR ATTENTION HAS TO BE GIVEN WHERE CHILDREN AND VULNERABLE PEOPLE ARE PRESENT.**
- THE USE OF ANY ACCESSORY OR ATTACHMENTS NOT RECOMMENDED BY COOPERS OF STORTFORD MAY CAUSE INJURIES AND INVALIDATE ANY WARRANTY YOU MAY HAVE.
- DO NOT LET THE POWER CABLE HANG OVER THE EDGE OF A TABLE OR COUNTER, OR TOUCH HOT SURFACES.
- DO NOT PLACE ON OR NEAR HEAT SOURCES.
- USE ON A LEVEL, STABLE SURFACE.
- DO NOT COVER THE ITEM IN ANY WAY WHEN IN USE AS THIS MAY CAUSE OVERHEATING.
- DO NOT USE THIS APPLIANCE FOR ANY OTHER USE THAN THE INTENDED USE IT IS DESIGNED FOR.
- ENSURE THERE IS SUFFICIENT DISTANCE BETWEEN THIS APPLIANCE AND OTHER ITEMS ON THE WORK SURFACE OR WORK STATION, OR SUFFICIENT DISTANCE BETWEEN THE APPLIANCE AND THE CONFINES OF THE AREA IN WHICH IT IS SITUATED. WE RECOMMEND A MINIMUM CLEARANCE AROUND ALL FACES OF AT LEAST 10CM.
- KEEP ITEM AT LEAST 1M AWAY FROM COMBUSTIBLE MATERIAL SUCH AS FURNITURE, CUSHIONS, BEDDING, PAPER, CLOTHES, CURTAINS ETC.
- TO PROTECT AGAINST ELECTRIC SHOCK, DO NOT IMMERSE POWER CABLE, MAINS ADAPTOR/ CHARGER OR APPLIANCE, IN WATER OR ANY OTHER LIQUID.
- UNPLUG FROM OUTLET WHEN NOT IN USE AND BEFORE CLEANING.
- DO NOT OPERATE ANY APPLIANCE WITH A DAMAGED POWER CABLE OR MAINS ADAPTOR/ CHARGER OR AFTER THE APPLIANCE MALFUNCTIONS, OR HAS BEEN DAMAGED IN ANY MANNER. WE RECOMMEND THAT THE POWER CABLE IS NOT KINKED OR STORED IN A TWISTED MANNER, AND THAT YOU REGULARLY CHECK ITS CONDITION.
- DO NOT LEAVE UNATTENDED.
- ALWAYS SWITCH OFF AFTER COOKING OR IF YOU HAVE TO LEAVE THE ROOM.
- THE USE OF OVEN GLOVES IS RECOMMENDED WHEN HANDLING HOT FOOD OR HOT APPLIANCES.
- ALLOW TO COOL BEFORE CARRYING OR PLACING IN STORAGE.
- ANY REPAIRS MUST BE CONDUCTED BY A QUALIFIED ELECTRICIAN OR ELECTRICAL SERVICE AGENT.

PLUG WIRING (UK & IRELAND)

- This appliance is supplied with a mains adaptor/charger with BS1363 plug pins.
- For Europe, the 2-pin plug is the VDE EN 50525 type.
- **DO NOT** use if the socket outlets in your home or office are not suitable for this product's mains adaptor/ charger plug pins. Attempts to insert the mains adaptor/charger into the wrong socket is likely to cause electric hazard or damage.
- **DO NOT** use with any other mains transformer/charger than the one that is supplied with this product.

CLEANING AND CARE INSTRUCTIONS:

- Always disconnect the plug from the mains before cleaning!
- Clean the device and the silicone pads with a damp cloth and mild detergent. Do not use abrasive or corrosive cleaning agents. The silicone pads can be removed to make cleaning easier.
- Never allow the appliance to come into contact with water or other liquids.
- Do NOT immerse in water. Risk of electric shock.
- Do NOT use abrasive cleaners or sharp utensils to clean this appliance.

- Wipe surfaces with a soft dry cloth.
- Allow to dry thoroughly before further use.
- Store the appliance in a cool, dry place.

DISPOSAL

- Coopers of Stortford use recyclable or recycled packaging where possible.
- Please dispose of all packaging, paper, cartons in accordance with your local authority recycling regulations.
- At the end of the product's lifespan please check with your local council authorised household waste recycling centre for disposal.

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UK: The Enterprise Department Ltd.